

KN95 Mask (Copy)

SKU: MP31971



Categories: [Masks](#)



Description

140.00 mm

220.00 mm



KN95 MASK

FFP2 PROTECTION RESPIRATOR

INSTRUCTIONS



1. Clean your hands with soap and water or hand sanitizer before touching the mask.
2. Remove a mask from the box and Determine which side of the mask is the top. The side of the mask that has a stiff bendable edge is the top and is meant to mold to the shape of your nose.
3. Face Mask with Ear loops: Hold the mask by the ear loops. Place a loop around each ear.
4. Mold or pinch the stiff edge to the shape of your nose.
5. Pull the bottom of the mask over your mouth and chin.

[Product name]
CELECARE Protective Mask

STERILE R
Sterilized by Gamma Radiation

[Check Before Using]

1. Make sure there are no obvious tears or holes in either side of the mask;
2. No damage to the ear loops and with good elasticity;
3. No damage or break in the stiff bendable nose clip.


[User Seal Check]

1. Don the face mask and adjust so it is snug but not overly tight;
2. Gently breathe in so the mask collapse slightly, and hold your breath for 10 seconds;
3. If it remains slightly collapsed while you hold your breath, this indicates that there is no inward leakage of air and the mask is sufficiently sealed for use.

[Using Scope]

For adults;
Non washable;
It should be stored at -20°C-30°C and less than 80% humidity;
Validity period: 3 years

[Date of revision of the manual]
March 30, 2020

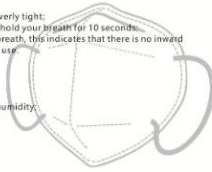

FDA CE EN 149:2001+A1:2009 ISO13485

CELECARE
CELECARE PROTECTIVE MASKS
CELECARE PROTECTIVE MASKS
CELECARE PROTECTIVE MASKS

Manufactured by: (Sheng) Top Quality PPE Co., Ltd.
Address: No. 100, Xiangyin Road, Xiangyin Town, Wenzhou City, Zhejiang Province, China
Tel: +86 577 86666666
www.celecure.com